

4 FT – 7 FT TELESCOPING ALUMINUM WHEELCHAIR RAMP

TMR7B
320017



TITAN
RAMPS

Operator's Manual



Read the Operator's Manual entirely. When you see this symbol, the subsequent instructions and warnings are serious follow without exception. Your life and the lives of others depend on it!

PARTS DIAGRAM / EXPLODED VIEW



KEY	DESCRIPTION	QTY
(1)	RAMP	2
(2)	CARRY BAG	1

SAFETY INFORMATION

MAXIMUM *OCCUPIED/UNOCCUPIED HEIGHT LIMIT TO GROUND LEVEL:**

- 2 FT Ramp: Occupied loading: 4-in maximum, unoccupied loading height: 6-in.
- 3 FT Ramp: Occupied loading: 6-in maximum, unoccupied loading height: 9-in.
- 4 FT Ramp: Occupied loading: 8-in maximum, unoccupied loading height: 12-in.
- 5 FT Ramp: Occupied loading: 10-in maximum, unoccupied loading height: 15-in.
- 6 FT Ramp: Occupied loading: 12-in maximum, unoccupied loading height: 18-in.
- 7 FT Ramp: Occupied loading: 14-in maximum, unoccupied loading height: 21-in.
- 8 FT Ramp: Occupied loading: 16-in maximum, unoccupied loading height: 24-in.
- 9 FT Ramp: Occupied loading: 18-in maximum, unoccupied loading height: 27-in.
- 10 FT Ramp: Occupied loading: 20-in maximum, unoccupied loading height: 30-in.

*Occupied is a person sitting in the wheelchair/scooter while loading (2:12 slope or 9.6-degree angle)

**Unoccupied is loading with nobody sitting in the wheelchair/scooter (3:12 slope or 14.5-degree angle)

MAXIMUM WEIGHT CAPACITY IS 600 LB

(WEIGHT RATING IS BASED ON EVENLY DISTRIBUTED LOAD ON 2 AXLES)

ASSEMBLY INSTRUCTIONS

STEP 1

No tools are required to assemble this product.

(However, if the ramp is to be secured to the landing surface, then appropriate tools will be required). Remove items from packaging, Check for damage.

STEP 2

Press the button and pull out the telescopic part until clicks into place, to extend the ramp, hold it horizontally or lay it on the floor. Note: If the latch is released (or is not engaged) when the ramp is not horizontal, the inner section may slide out under its own weight. At full extension, the latch will engage to prevent the inner ramp sliding out totally



STEP 3

Place the ramp on the landing surface. Ensuring that the top lip is resting securely on the landing surface.

ACKNOWLEDGEMENT OF RISK AND RELEASE OF LIABILITY

The use of any equipment, including this one, involves the potential risk of injury. Apart from any warranty claim that might be presented for a claimed defect in material or workmanship of the product, you accept and assume full responsibility for any and all injuries, damages (both economic and non-economic), and losses of any type, which may occur, and you fully and forever release and discharge Titan, its insurers, employees, officers, directors, associates, and agents from any and all claims, demands, damages, rights of action, or causes of action, present or future, whether the same be known or unknown, anticipated, or unanticipated, resulting from or arising out of the use of said equipment.

This equipment must be used with care by capable and competent individuals under supervision, if necessary.

TITAN LIMITED WARRANTY: TERMS, EXCLUSIONS AND LIMITATIONS OF REMEDIES

This product comes with a one (1) year limited warranty that can be found at www.titanramps.com/warranty.html. Please review the same for all details regarding the Titan Limited Warranty.

THE TITAN LIMITED WARRANTY FOUND AT [WWW.TITANRAMPS.COM/WARRANTY.HTML](http://www.titanramps.com/warranty.html) IS EXCLUSIVE AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE, EACH OF WHICH IS HEREBY DISCLAIMED.