



Aluminum Adjustable Threshold Ramp

Instructions for Part # APT32 and APT49



PARTS

Part	Description	Qty.
1	Ramp	1
2	Leg	2
3	Nut	4
4	Spacers	4



ASSEMBLY

Step 1. Measure the threshold to determine how high the ramp legs need to be adjusted.



Step 2. To install the ramp legs, add 2 spacers to the top and bottom of each leg and adjust each leg to the proper height. With one nut threaded onto the leg, place the leg in the hole in the lip at the top of the ramp. Install the second nut onto the leg on the other side of the lip. Keep the nuts threaded away from the ramp lip to adjust the leg height. When the leg is at the proper height, securely tighten both nuts against the ramp lip using a wrench.

Step 3. Place the ramp against the threshold, ensuring it is stable and at the proper height.



GENERAL GUIDELINES & SAFETY

- It is the user's responsibility to read and follow all instructions
- Keep these instructions with the product at all times and review before each use
- It is the responsibility of this product's owner to furnish the instructions to any person that borrows or purchases the product
- Inspect the product before use to ensure it is assembled properly and all parts are in safe working order and free of defects
- Never modify this product in any way
- All circumstances cannot be addressed in these instructions
- Please use common sense and practice general safety measures when using this product
- Parts and/or instructions are subject to change without notice
Stop using ramp system if it becomes damaged or unstable.
- Only load and unload onto a stable, level surface.
- Never exceed the maximum weight limit of 600 lbs.
- Use caution at all times when loading and unloading.
- Ramp surfaces will have less traction when wet, use caution.
- Always make sure the top ramp lip is secured onto step, landing, or vehicle before use.
- Always follow equipment manufacturer's recommendations for proper degree of incline/decline.
- Always use ramp with qualified helper and lap belt.

PRODUCT WARRANTY & LIABILITY

GENERAL PRODUCT WARRANTY: Products purchased from the Authorized Dealer (original place of purchase) or Merchant ("Dealer") will be free of defects in material and workmanship at the time of receipt, and will meet the specifications stated at the place of purchase transaction or online at the Dealer's website, under normal use and service when correctly installed, operated and maintained. This product warranty is effective for the period of time stated below, unless otherwise stated in the product instructions or depicted in the product advertisement. All Authorized Dealer warranties are NON-TRANSFERABLE and cover only the original end purchaser. This limited warranty does not cover products purchased through non-authorized dealers. Non-authorized dealer receipts are not accepted for warranty verification.

*****WARRANTY CLAIMS MUST BE MADE DIRECTLY TO THE ORIGINAL PLACE OF PURCHASE.*****

WARRANTY PERIOD: This warranty remains in force for one year from the date of the product's accepted delivery. The Authorized Dealer offers a one year manufacturer's warranty for most products unless otherwise specified on the product advertisement.

WARRANTY SERVICE: The Authorized Dealer will replace any defective or malfunctioning part at no charge, including payment of the shipping costs of parts or replacement product to and from the manufacturer. The purchaser is responsible for labor charges. If the product does not meet specifications as depicted in the advertisement, the Authorized Dealer will refund the full purchase price of the product.

Questions regarding the warranty on a specific product and warranty claims should be directed to the Authorized Dealer with whom the purchase transaction was made.

WARRANTY LIMITATIONS: The above warranty does not apply to products that are repaired, modified or altered by any party other than the Authorized Dealer; are subjected to unusual physical stress or conditions (such as overloaded ramps or corrosion), natural disaster, governmental actions, wars, strikes, labor disputes, riots, theft, vandalism, terrorism or any reason beyond reasonable control; are damaged due to improper installation, misuse, abuse, accident or negligence in use, storage, transportation or handling, or tampering; or to products that are considered consumable items or items requiring repair or replacement due to normal wear and tear.

Product should be inspected prior to signing for delivery. Product damage incurred during shipping, unless noted on the Bill of Lading at the time of delivery, renders this warranty void.

LIMITED LIABILITY: In no event shall **THE AUTHORIZED DEALER** be liable to the purchaser or any third party for any indirect, incidental, consequential, special, exemplary or punitive damages arising out of the use of the product, including, without limitation, property damage, loss of value of the product or any third party products that are used in or with the product, or loss of use of the product or any third party products that are used in or with the product.